

Biocut-C

Reusable automatic device
for soft tissues biopsy

Code: Biocut-C

A



B



Shooting
button

C



Piston to
charge the
cannula

D



Piston to
charge the
stylet

E



Easy and safe
To be used
with
BE needle



CE

biomedical



Biocut-C

Reusable automatic device
for soft tissues biopsy

Code: Biocut-C

Instructions of use:



Open the cover
and insert BE needle
into the gun


PUSH




Push the shooting button



Close the cover
Withdraw the piston
on the left to charge
the cannula

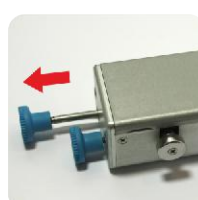

Withdraw
the piston on the left
to discharge
the sample

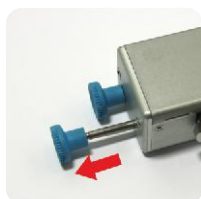


Withdraw the piston
on the right to
charge the stylet

To order:

CODE
BIOCUT C

PACKAGE
I

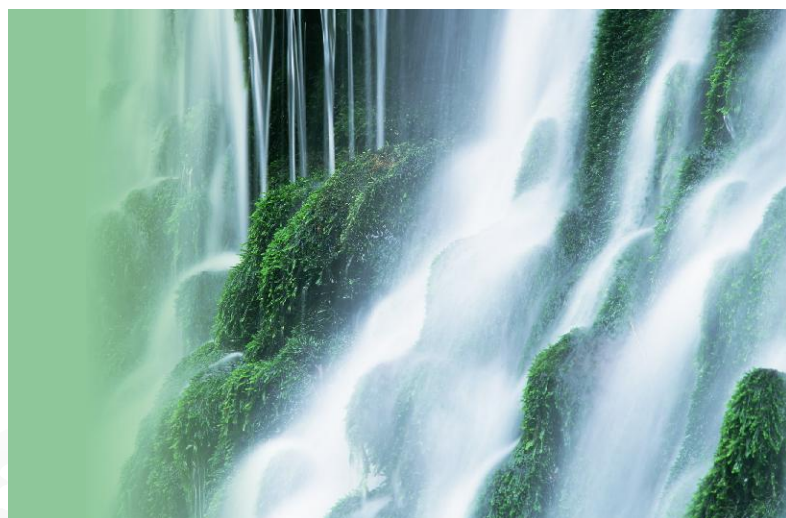

biomedical

Biomedical srl Via G.B. Lulli, 43 50144 Firenze Italy
Tel +39 055 352678 +39 055 332822 Fax +39 055 331629
www.biomedical-bio.net sales@biomedical-bio.net

Bone biopsy needles
Guillotine needles for biopsy on soft tissues
Cyto-aspiration needles
Different specialistic needles and sets
Loco-regional anaesthesia needles and sets
Thermic and absorbing sheets