
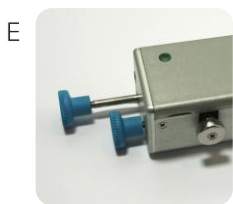
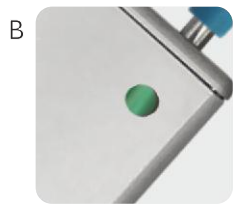


Biocut-M

Reusable automatic device
for soft tissues biopsy

Code: Biocut-M

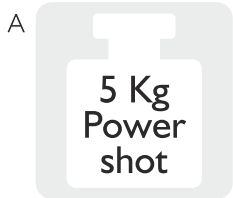

CE

biomedical



Biocut-M

Reusable automatic device
for soft tissues biopsy

Code: Biocut-M

Instructions of use:



Open the cover
and insert PG needle
into the gun



When the indicators
are green
Bioshot-M is charged




Close the cover
Withdraw the piston
on the left to charge
the cannula


PUSH



Push the shooting button

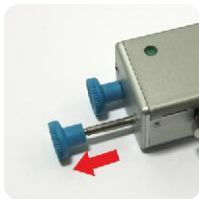

Withdraw the piston
on the right to
charge the stylet

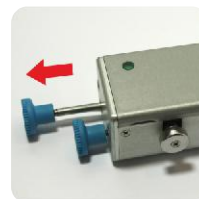


Withdraw
the piston on the left
to discharge
the sample

To order:

CODE
BIOCUT M

PACKAGE
I

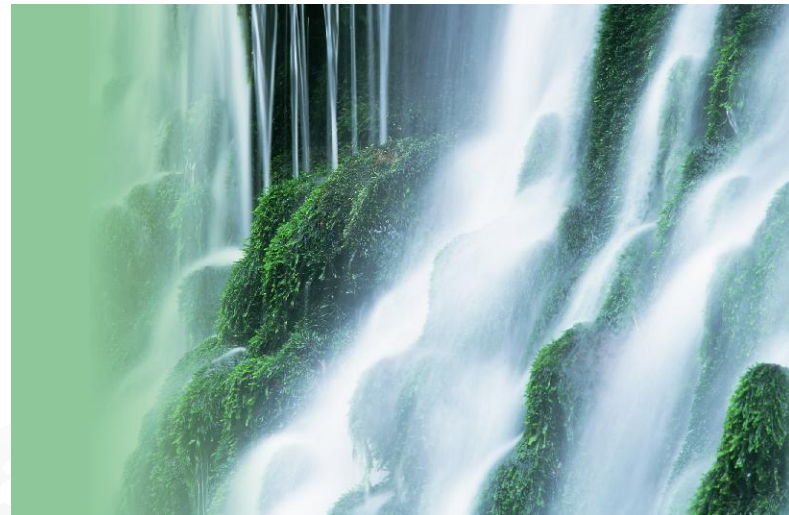

biomedical

Biomedical srl Via G.B.Lulli, 43 50144 Firenze Italy
Tel +39 055 352678 +39 055 332822 Fax +39 055 331629
www.biomedical-bio.net sales@biomedical-bio.net

Bone biopsy needles

Guillotine needles for biopsy on soft tissues

Cyto-aspiration needles

Different specialistic needles and sets

Loco-regional anaesthesia needles and sets

Thermic and absorbing sheets