

Safety catch



Lower releaser for automatic shot



Upper releaser for automatic shot



Cursor to charge the stylet



Cursor to charge the cannula



Choice between two different sampling depths



Sliding stopper Internal echo-marker Depht marks



G


Charge indicators





Bioshot

Disposable automatic guillotine needle for biopsy on soft tissues

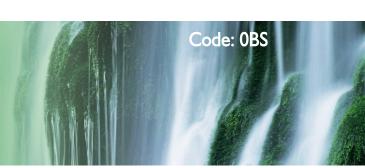

Istruction of use



Withdraw the cannula cursor until charged, both it stops (hooking click) Withdraw the stylet cursor until it stops (hooking click)



When it is indicators are colored.



Choose the sample's lenght to take by moving the proper selector on the longer or the shorter position.



Put the safety catch on the LOAD position.



Push the upper shooting botton*



Or artenatively push the lower shootin button*



To estract the needle: withdraw the cannula cursor until it stops (hooking click) in òrder to uncover the stylet for the frustule taking

(*) The device can be used also as semi-automatic needle pushing the shooting button softly:







The second one starts the cannula.

If the indicators are not colored the device is uncharged.

To order:

CODE	SIZE	PACKAGE	C. CODE
0BS 1410	I4GxI0 cm	20	DARK GREEN
0BS 1415	I4GxI5 cm	20	DARK GREEN
0BS 1420	I4Gx20 cm	20	DARK GREEN
0BS 1610	I6GxI0 cm	20	WHITE
0BS 1615	I6GxI5 cm	20	WHITE
0BS 1620	16Gx20 cm	20	WHITE
0BS 1625	16Gx25 cm	20	WHITE
0BS 1810	18Gx10 cm	20	PINK
0BS 1815	18Gx15 cm	20	PINK
0BS 1820	18Gx20 cm	20	PINK
OBS 1825	18Gx25 cm	20	PINK



On request Biomedical is available to provide needles with different lenghts and 20G diameter.

biomedical

Biomedical srl Via G.B.Lulli, 43 50144 Firenze Italy Tel +39 055 352678 +39 055 332822 Fax +39 055 331629 www.biomedical-bio.net sales@biomedical-bio.net

Bone biopsy needles Guillotine needles for biopsy on soft tissues Cyto-aspiration needles Different specialistic needles and sets Loco-regional anaesthesia needles and sets Thermic and absorbing sheets